

HEARING AID MANUAL

How to Start Using the Hearing Aid?

1. Install batteries:
 - a. Open up the back cover by using a screw driver and place two AAA batteries into the battery box. Make sure to place the batteries according to the terminals displayed on the battery box.
 - b. Close the cover
2. Make sure that the two controls are turned off properly.
3. Connect the headphones and microphones to their respective jacks.
4. Put the R headphone in the right ear and the L headphone in the left ear.
5. Place the R microphone on the right side of your body and the L microphone on the left side of your body.
 - a. For getting the best output :
 1. Do not place microphones close to headphones, below your shoulders works best
 2. There should be a gap of at least 6 inches between the two microphones.
6. Turn on controls one at a time and then turn the volume up until the sound level is loud enough for you to hear sound.

Warning: Do not turn the volume all the way up before putting the headphones into your ears; this may cause minor hearing loss.

Troubleshooting

- Change batteries if sound is getting weaker
- Check the connections and make sure that the headphones and microphones are plugged tightly in their respective jacks.
- In case of any other problem, contact the respective senior design team through the following information:

sddec310@iastate.edu

This document was created with Win2PDF available at <http://www.daneprairie.com>.
The unregistered version of Win2PDF is for evaluation or non-commercial use only.